


LogicMark

GUARDIAN ALERT 911

Synching your Pendant and Base Station together so that they will communicate.

What to do: **READ ALL OF THE DIRECTION FIRST.**

1. Make sure the base station is connected to an electrical outlet and phone jack.
2. Press the “Set-Up” button in the Base Station until the red “Scanning” light comes on. Release the “Set-Up” button.
3. On the pendant, press and hold both the Panic Button (Blue) and the Test Button (Gray on bottom of pendant) at the same time. The “Scanning” light on the Base Station will turn off when synchronized.

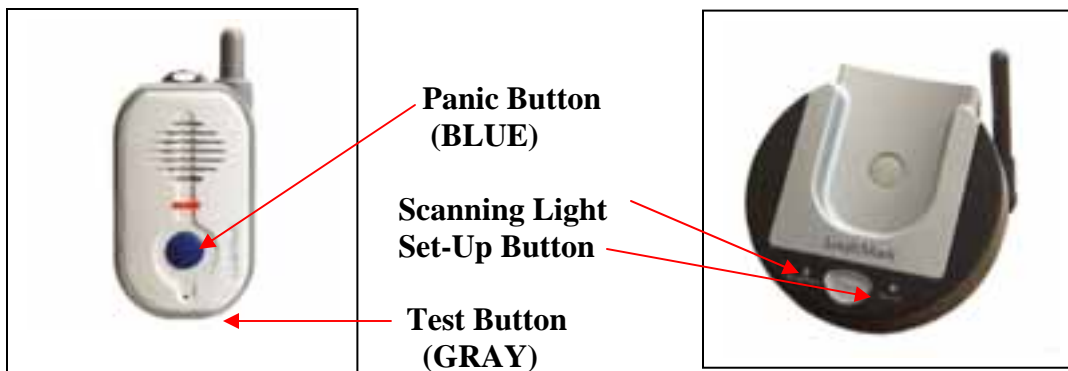

What to expect:

The red (Scanning) light will turn off on the Base Station. This will happen very fast, so make sure to be looking at the “Scanning” light before you push and hold both of the buttons described above.

At this point the Pendant and Base Station are synchronized.

Final Check:

You can now either push the BLUE Panic button and contact 911 to test the system or, press the GRAY test button on the bottom for 5 seconds for a system check.